## TEST PROCEDURE

**WARNING:** if purchasing a multipack, check that the sterile swab, the sterile tongue depressor, the test tube and the dropper have not been previously used (these disposable materials cannot be used twice).

- 1) Wash your hands with hot water and soap, rinse with clean water and dry.
- 2) Lay out the work tray on a clean, dry, flat surface. Place the empty plastic test tube in one of the compartments in the work tray. –**FIG. A**
- 3) Collecting the sample:
  - Open the package containing the tongue depressor, leaving it inside the package and next to you.
  - Open the package containing the sterile swab. **Avoid touching the cotton tip** and remove the swab using the plastic handle. –**FIG. B**
  - Stand in front of a mirror, tilt your head backwards and open your mouth as much as possible.
  - Pick up the tongue depressor with one hand and flatten the tongue.
  - Use the other hand to place the swab close to the throat. Touch the back of the throat - the area around the tonsils<sup>3</sup> and any reddened or painful part (soft palate<sup>1</sup>, uvula<sup>2</sup>) with the cotton tip. **Rotating** the swab is recommended because it increases the amount of collected sample. **If you are struggling, ask someone to help you collect the sample.** –**FIG. D**
- 4) After collecting the sample, insert the swab's cotton tip into the supplied plastic test tube that you had previously placed in the workstation. –**FIG. E**
- 5) Open the vial STREP A TEST - R1 by **unscrewing only the blue cap** (leave the white part firmly sealed), then add **4 drops** into the test tube and close it with the cap. –**FIG. F**
- 6) Next, open the vial STREP A TEST - R2 by **unscrewing only the green cap** (leave the white part firmly sealed), then add **4 drops** into the test tube and close it with the cap. –**FIG. G**
- 7) Holding the swab's plastic handle, rotate the swab against the sides of the test tube about 10 times to thoroughly mix the solution. Leave the swab to incubate for 2-5 minutes. –**FIG. H**
- 8) At the end of the incubation period, remove the test tube (with the swab and extraction fluid still in it) from the workstation. Using your thumb and index finger, press the sides of the test tube to release as much fluid as possible from the swab's cotton tip and collect it in the test tube. Remove the swab. **FIG. I** Dispose of the swab in compliance with local laws and put the test tube back into one of the compartments in the workstation.
- 9) Add the supplied dropper to the plastic test tube. –**FIG. L**
- 10) Dispense 1 drop into the well in the test case. –**FIG. M**  
*Note:* If the dispensed drop contains air bubbles, add another drop to the well.
- 11) Read the results after 5 minutes (some positive results can be seen after just 1 minute). Do not read after 10 minutes.

## RESULTS INTERPRETATION

**READ THE RESULT AFTER 5 MINUTES.  
DO NOT READ THE RESULT AFTER 10 MINUTES.**

## POSITIVE RESULT

Two coloured bands appear in the reading window by the T (test) and C (control) signs. The T band may be less intense (lighter) than the C line. **This result means that the test has detected the presence of Streptococcus A specific antigens in the sample. You should consult a physician.**

5 min.

## NEGATIVE RESULT

A coloured band appears only under the C (control) sign. **This result is negative and means that no Streptococcus A antigens have been detected in the sample from the pharynx or that their amount is too low to be detected.**

## NON VALID RESULT

No bands appear or there is a line only under the T (test) sign and not under the C (control) sign. In this case it is not possible to interpret the result of the test, which must be considered not valid. Repeat the test with a new sample.

## F.A.Q. – QUESTIONS AND ANSWERS

**HOW DOES THE STREP A TEST WORK?** Group A β-hemolytic Streptococcus could be involved in infections of the pharynx. For appropriate antibiotic treatment to be administered, the bacterium must be identified first. STREP A TEST can detect Streptococcus A antigens through the specific antibodies it contains. **WHEN CAN THE TEST BE USED?** STREP A TEST can be carried out if there are symptoms such as pain when swallowing, sore throat, red and swollen tonsils (sometimes with white spots or pus), small red patches on the back of the palate (soft or hard), swollen lymph nodes, fever, headache, nausea or vomiting, especially in children.

**CAN THE RESULT BE INCORRECT?** The result is correct if the instructions are followed carefully. However, the result may be incorrect if STREP A TEST is wet before performing the test or if the sample from the pharynx is not collected and prepared correctly (carefully follow the instructions for collecting the sample from the pharynx –fig.D). Incorrect results may also be obtained if the test is carried out after treatment with antibiotics or at the beginning of the infection (the concentration of bacteria is below the level of detection).

**HOW DO I INTERPRET THE TEST IF THE COLOUR AND INTENSITY OF THE TEST AND CONTROL BANDS ARE DIFFERENT?** The colour and the intensity of the bands are not important for the interpretation of the result. The bands must be full and homogeneous. The test is

positive regardless of the intensity of the colour of the Test band, even if it is weak.

**WHAT IS THE PURPOSE OF THE CONTROL BAND?** The appearance of this band means the test has worked correctly.

**WHAT SHOULD I DO IF THE RESULT IS POSITIVE?** If the result is positive, Streptococcus A has been detected in the sample from the pharynx and you must consult a physician. Please be aware the STREP A TEST cannot distinguish between live and dead bacteria. Patients who have recently recovered from Streptococcus A or similar infections may test positive because of inactive bacteria that are still present in the body.

**WHAT SHOULD I DO IF THE RESULT IS NEGATIVE?** A negative result means that the symptoms were not caused by the Streptococcus A or that the sample did not contain a concentration of bacteria that was high enough to be detected by the test. If symptoms persist, seek medical advice.

**IS THE RESULT RELIABLE IF READ AFTER 10 MINUTES?** No. The test must be read 5 minutes after the procedure is completed and no later than 10 minutes from this time.

**HOW ACCURATE IS THE STREP A TEST?** The test is very accurate. Assessment reports show a concordance rate above 95% (CI 88.25% – 100%) with reference methods. Although this test is reliable, it may yield false positive and negative results.

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